



Offering Medical Furniture & Supplies Since 1987

Buy Direct From The Manufacturer! ...and ~~save!~~



contact us online
Call 800.749.0933

Blood Draw Adjustable Chairs

Models:

1202-LXL, 1202-LXL/XT, 1202-SXXL, 1202-SXXL/XT, 1202-LTUT, 1202-LTU,
1202-LTT, 1202-LT

Assembly/Usage Instructions

Install Arms-

- Insert arm posts into chair frame. Place flip-up arm on right side of chair. Place straight arm on left side of chair. Insert adjuster knob into threaded hole on chair frame.

Usage Instructions

Arm Adjustment-

- To adjust armrests loosen the adjuster knob $\frac{1}{4}$ turn and move the arm up or down and retighten the adjuster knob.

Custom Comfort Medtek
3939 Forsyth Road, Suite A • Winter Park, FL 32792

Mailing Address
PO Box 4779 • Winter Park, FL 32793

Phone: (407) 332-0062 • Toll Free: (800) 749-0933 • Fax: (407) 332-0025
Toll Free Fax: (800) 842-7718