



## Blood Draw Adjustable Chairs

Models:  
1202-LAH, 1202-LU/AH

### Assembly/Usage Instructions

#### Attach Seat-

- When chair is equipped with a molded plastic seat- place the seat onto chair frame. Tighten seat to frame with the supplied nuts.
- When chair is equipped with upholstered seat and back- place chair face down and mount upholstered backrest to chair frame using the supplied stainless steel bolts. (Only use stainless steel bolts, regular steel bolts could stretch causing the chair back to loosen.) Place the seat on the frame with rounded corners facing the front. Bolt the seat down with the supplied bolts.
- Place hydraulic base on floor and place frame on to hydraulic lift post. The slot on the seat bracket located on the bottom of the chair frame must be placed on to the pin of the hydraulic lift post.

#### Install Arms-

- Insert arm posts into chair frame. Place flip-up arm on right side of chair. Place straight arm on left side of chair. Insert adjuster knob into threaded hole on chair frame.

### Usage Instructions

#### Arm Adjustment-

- To adjust armrests loosen the adjuster knob  $\frac{1}{4}$  turn and move the arm up or down and retighten the adjuster knob.

#### Height Adjustment-

- To adjust height of chair place foot on hydraulic lift pedal and use short pumps until chair is raised to proper height. Caution: when raising the chair depress the foot pedal  $\frac{3}{4}$  of the way down, the final  $\frac{1}{4}$  will release the pressure lowering the chair back down prematurely.
- To keep the chair from rotating you can apply the brake by lifting up on lift pedal with the top of your foot.
- To lower chair place foot on lift pedal and depress pedal. The weight of the patient will slowly lower the chair.