



Reclining Chair Series

Models:

BA1508, BA1508-XT, BA1508-XL, BA1508-XL/XT, BA1508-AL/FT

Assembly/Usage Instructions

Reclining the Chair-

- Make sure casters are fully locked when reclining chair.
- Chair may be placed in regular recline positions by pulling the gray grab bar located on back of chair. When chair is being reclined using the grab bar, the footrest will also be engaged to elevate the feet.
- To recline the back of chair further or to place into Trendelenburg position, use black release handle located on grab bar. Squeeze the black handle until back is lowered to desired position.
- To elevate back to upright position use grab bar to raise the back to the regular recline position. Use black release handle to further elevate back to proper position.

Installing the Optional Auxiliary Arms-

- Attach one arm adjuster on each side of chair using a 7/16" socket or wrench.
- Insert arm posts into arm adjuster. If chair has the optional "flip-up" arm, place flip-up arm on right side of chair. Place straight arm on left side of chair.

Adjusting the Optional Auxiliary Arms-

- To adjust armrests, loosen the black adjuster handle and move the arm up/down or left/right. Re-tighten handle to secure arm into position. The black adjuster handle position can be adjusted by pressing in gray handle button and rotating handle.
- Adjust angle of the swing arm by rotating adjustment knob underneath armrest.

Installing the Optional Side Trays-

- Attach folding tray bracket to either the left, right or both sides of the chair using a 1/16" socket or wrench.

Operating the Optional Tray-

- To raise trays lift tray up, pull forward then back until pins on each side of tray go into holders. To lower side trays pull forward and up until pins clear holders.