

Adjustable Reclining Chair Series

Models: BA1556-AP

Assembly/Usage Instructions

Attach Arm Adjusters-

- Attach one arm adjuster on each side of chair with a 7/16" socket or wrench.

Install Arms-

- Insert arm posts into arm adjuster. If chair has the optional "flip-up" arm, place flip-up arm on right side of chair. Place straight arm on left side of chair.

Battery Installation-

- Charge battery for 24 hrs prior to first use. Future charges will take proximately 3 hrs.
- Install the battery charger on the wall using the supplied screws and or anchors. Once plugged in green indicator light will come on.
- Squeeze the release bracket on the top of the battery and snap into charger. Orange charging light will come on to indicate charging. Light will go off once battery has been fully charged. Use the release lever to remove battery from charger.
- Squeeze the release bracket on the top of the battery and snap onto control bracket mounted on back of chair.

Usage Instructions

Reclining the Chair-

- Lock casters before attempting to recline chair.
- Step on foot assist bar and pull back on grab bar at rear of chair. When chair is being reclined, the footrest will also be engaged to elevate the feet.
- To place into Trendelenburg position, squeeze the black release handle located on grab bar and continue to lower the backrest.

Return Backrest to Upright Position-

- **First-** using the grab bar, raise back to the regular recline position. **Second-** use black release handle to further elevate back to proper position.

Height Adjustment-

- Depressing the hand held remote up and down arrows will raise and lower the chairs seat height. Make sure chair is fully flowered prior to entering or exiting the chair.