

Adjustable Reclining Chair Series

Models:
BA1556

Assembly/Usage Instructions

Attach Arm Adjusters-

- Attach one arm adjuster on each side of chair with a 7/16" socket or wrench.

Install Arms-

- Insert arm posts into arm adjuster. If chair has the optional "flip-up" arm, place flip-up arm on right side of chair. Place straight arm on left side of chair.

Install Foot Pedal-

- Insert foot pedal into holes on hydraulic pump. Foot pedal should be on left side of chair as sitting.

Usage Instructions

Reclining the Chair

- Lock casters before attempting to recline chair.
- Step on foot assist bar and pull back on grab bar at rear of chair. When chair is being reclined, the footrest will also be engaged to elevate the feet.
- To place into Trendelenburg position, squeeze the black release handle located on grab bar and continue to lower the backrest.

Return Backrest to Upright Position-

- **First-** using the grab bar, raise back to the regular recline position. **Second-** use black release handle to further elevate back to proper position.

Height Adjustment-

- To adjust height of chair place foot on hydraulic lift pedal and use short pumps until chair is raised to proper height. *Caution: when raising the chair depress the foot pedal $\frac{3}{4}$ of the way down, the final $\frac{1}{4}$ will release the pressure lowering the chair back down prematurely.*
- To lower chair place foot on lift pedal and depress pedal. The weight of the patient will slowly lower the chair's height.

Install Optional "Option Adapter"

Optional A1556-AD Option Adapter-

- Remove all four bolts from rear caster and replace with the longer bolts.
- Place adapter on top of caster plate and align holes. Replace bolts and nuts with the supplied longer ones and tighten.
- Secure optional accessory to adapter at desired height.