





Important

Please read these instructions before installing, maintaining, operating or using this equipment otherwise injury and/or damage may result

Visually inspect equipment prior to putting it into use. Keep hands and clothing away from all moving parts. Always observe recommended weight capacity. If equipment is equipped with locking casters, be sure to lock all casters before using.

Cleaning Instructions

- All metal parts can be cleaned with a clean damp cloth. More stubborn stains may require a mild soap and water mixture. Dry completely using a soft cloth.
- To ensure years of lasting durability for your vinyl items, we suggest that you inspect your upholstery on a regular basis to ensure maximum upkeep.
- If a stain or discoloration should appear on your vinyl, clean immediately.
- To remove ordinary stains, we suggest you use a mild soap and water mixture. After cleaning, wipe soiled area clean with a damp cloth. Dry completely using a soft, lint-free cloth.
- For more difficult stains, we suggest a diluted mixture of water and 10% bleach. You should follow the general cleaning procedure with any of these special cleaners.
- As with any cleaning agents, you should use in an inconspicuous spot before using it on the original stain.

Custom Comfort Medtek 3939 Forsyth Road, Suite A • Winter Park, FL 32792

Mailing Address
PO Box 4779 • Winter Park, FL 32793

Phone: (407) 332-0062 • Toll Free: (800) 749-0933 • Fax: (407) 332-0025 Toll Free Fax: (800) 842-7718