

## Adjustable & Free Standing Infant Draw Stations

Models:

MA3014-AH, MA3014-FS, MA3091-AH, MA3091-FS

### Assembly Instructions

#### ***Install Tray-***

- Fasten tray and bracket to end of bed frame with 1/4" x 1 3/4" bolts, washers and lock nuts.

*(Standing Infant Stations)*

#### ***Install Legs-***

- Fasten legs to inside of brackets on end of bed frame with 1/4" x 1 3/4" bolts, washers and lock nuts.
- Fasten leg cross member to inside of legs with 1/4" x 1 3/4" bolts, washers and lock nuts.

*(Adjustable Infant Stations)*

#### **Install Frame-**

- Install frame on hydraulic base by lining up the key in the hydraulic shaft and the notch in the mounting plate on the bottom of the frame.

### Usage Instructions

Models:

MA3014-AH & MA3091-AH

#### **Height Adjustment-**

- To adjust height of station- place foot on hydraulic lift pedal and use short pumps until station is raised to proper height. *Caution: when raising the station depress the foot pedal 3/4 of the way down, the final 1/4 will release the pressure lowering the station back down prematurely.*
- Station will rotate in a circle. To apply brake, lift up on lift pedal with the top of your foot.
- To lower station place foot on lift pedal and depress pedal all the way to the bottom. The weight of the infant will slowly lower the station's height.