

Wall Mounted Infant Draw Stations

Models:

MA3014-WMV, MA3014-WMH, MA3091-WMV

Assembly Instructions

Install Tray-

- Fasten tray and bracket to end of bed frame with 1/4" x 1 3/4" bolts, washers and lock nuts.

Check for Clearance-

- Identify the best location for the installation. Hold the unit in place and open the bed. Make sure you have taken the operating clearance into consideration.

Based upon the type of wall on which you will be mounting the station, determine the best method of mounting. You may have spare hardware left over when you are finished with installation.

Mounting the Frame-

- The horizontal unit has two mounting holes provided in the center of the unit. If possible, locate the wood or metal wall studs for fastening the drawing station in the desired location.
- If the walls do not have studs on center, **make sure at least one side of the station is installed into the stud.** Use toggles (supplied) for holes where stud is not present.