



Offering Medical Furniture & Supplies Since 1987

Buy Direct From The Manufacturer! ...and  save!

[contact us online](#)
Call 800.749.0933

Recliner Chair Series

Models:
MC1227

Assembly/Usage Instructions

The chair is shipped completely assembled for you!

Usage Instructions

Reclining the Chair-

- If chair has casters, make sure casters are fully locked before reclining chair.
- Standing behind the chair, pull back on the black recline handles, the weight of the patient will recline the chair and hold it in position.
- The arms will follow the patient back to regular recline position.

Custom Comfort Medtek
3939 Forsyth Road, Suite A • Winter Park, FL 32792

Mailing Address
PO Box 4779 • Winter Park, FL 32793

Phone: (407) 332-0062 • Toll Free: (800) 749-0933 • Fax: (407) 332-0025
Toll Free Fax: (800) 842-7718