



Offering Medical Furniture & Supplies Since 1987

Buy Direct From The Manufacturer! ...and  save!

[contact us online](#)  
Call 800.749.0933

## Recliner Chair Series

Models:

VM5000-L, VM5000-LXT, VM5000-LXL, VM5000-LXL/XT

### Assembly/Usage Instructions

#### Install Arms-

- Insert arm posts into arm adjuster. If chair has the optional “flip-up” arm, place flip-up arm on right side of chair. Place straight arm on left side of chair.

### Usage Instructions

#### Reclining the Chair-

- If chair has casters, make sure casters are fully locked before reclining chair.
- Standing behind the chair, pull back on the black recline handles, the weight of the patient will recline the chair and hold it in position.
- The arms will follow the patient back to regular recline position.

#### Arm Adjustment-

- To adjust armrests loosen the black adjuster handle and move the arm up/down or left/right. Re-tighten handle to secure arm into position. The black adjuster handle position can be adjusted by pressing in gray handle button and rotating handle.
- Adjust angle of the swing arm by rotating adjustment knob underneath armrest.

Custom Comfort Medtek  
3939 Forsyth Road, Suite A • Winter Park, FL 32792

Mailing Address  
PO Box 4779 • Winter Park, FL 32793

---

Phone: (407) 332-0062 • Toll Free: (800) 749-0933 • Fax: (407) 332-0025  
Toll Free Fax: (800) 842-7718