



Offering Medical Furniture & Supplies Since 1987

Buy Direct From The Manufacturer! ...and  save!

[contact us online](#)
Call 800.749.0933

Surgeon's Stool

Models:
4080-xx

Assembly/Usage Instructions

Your surgeons stool comes completely assembled for you!

Usage Instructions

Height Adjustment-

- To adjust height of chair place foot on hydraulic lift pedal and use short pumps until chair is raised to proper height. *Caution: when raising the chair depress the foot pedal 3/4 of the way down, the final 1/4 will release the pressure lowering the chair back down prematurely.*
- Chair will rotate in a circle. To keep the chair from rotating you can apply the brake by lifting up on lift pedal with the top of your foot.
- To lower chair place foot on lift pedal and depress pedal. The weight of the patient will slowly lower the chair's height.

Custom Comfort Medtek
3939 Forsyth Road, Suite A • Winter Park, FL 32792

Mailing Address
PO Box 4779 • Winter Park, FL 32793

Phone: (407) 332-0062 • Toll Free: (800) 749-0933 • Fax: (407) 332-0025
Toll Free Fax: (800) 842-7718