

Blood Draw Chairs

Models:

1201-L, 1201-LXT, 1201-LU, 1201-LU/XT, 1201-LXL, 1201-LXL/XT, 1201-SXXL, 1201-SXXL/XT,
1201-LTUT, 1201-LTU, 1201-LTT, 1201-LT

Assembly/Usage Instructions

Insert Leg Levelers-

- Insert levelers into holes located on bottom of chair and adjust accordingly.

Attach Seat-

- When chair is equipped with a molded plastic seat- place the seat onto chair frame. Tighten seat to frame with the supplied nuts.
- When chair is equipped with upholstered seat and back- place chair face down and mount upholstered backrest to chair frame using the **supplied stainless steel bolts**. (*Only use stainless steel bolts, regular steel bolts could stretch causing the chair back to loosen.*) Place the seat on the frame with rounded corners facing out. Bolt the seat down with the supplied bolts using a 7/16" socket or wrench.

Attach Arm Adjusters-

- Attach one arm adjuster on each side of chair using a 7/16" socket or wrench.

Install Arms-

- Insert arm posts into arm adjuster. Place flip-up arm on right side of chair. Place straight arm on left side of chair.

Usage Instructions

Arm Adjustment-

- To adjust armrests loosen the black adjuster handle and move the arm up/down or left/right. Re-tighten handle to secure arm into position. The black adjuster handle position can be adjusted by pressing in gray handle button and rotating handle.
- Adjust angle of the swing arm by rotating adjustment knob underneath armrest.