

Hydraulic Reclining Lounge Chair

Model:
GA1972

Assembly/Usage Instructions

Install Arms-

- Remove locking nut and small washer from back of chair, place arm bracket on to studs leaving large washer between arm and chair. Place push-bar on the outside of the arm bracket, replace washer and lock nut (hand tighten only at this time).
- Place arm post into arm assembly - level armrests and tighten arm bracket fully with a 1/2" wrench. Adjust armrest to desired level and secure by tightening arm adjuster handle.

Usage Instructions

Height Adjustment-

- To adjust height of chair place foot on hydraulic lift pedal and use short pumps until chair is raised to proper height. *Caution: when raising the chair depress the foot pedal 3/4 of the way down, the final 1/4 will release the pressure lowering the chair back down prematurely.*
- To lower chair place foot on lift pedal and depress pedal. The weight of the patient will slowly lower the chair's height.

Recline Adjustment-

- Place hands on lift handle and squeeze black release handle. The lounge can be stopped and held in any position by releasing the release handle at the desired position.

Install Optional "Option Adapter"

Optional A1556-AD Option Adapter-

- Remove the four bolts and nuts from rear caster and replace with longer bolts.
- Place adapter on top of caster plate and align holes. Replace bolts and nuts with the supplied longer ones and tighten.
- Secure optional accessory to adapter at desired height.