

Blood Draw Chairs

Models:

1202-L, 1202-LXT, 1202-LU, 1202-LU/XT

Assembly/Usage Instructions

Insert Legs-

- Lay chair frame face down on floor. Line up arrows on chair frame and legs then insert legs into chair frame.

Attach Seat-

- When chair is equipped with a molded plastic seat- place the seat onto chair frame. Tighten seat to frame with the supplied nuts.
- When chair is equipped with upholstered seat and back- place chair face down and mount upholstered backrest to chair frame using the **supplied stainless steel bolts**. (*Only use stainless steel bolts, regular steel bolts could stretch causing the chair back to loosen.*) Place the seat on the frame with rounded corners facing the front. Bolt the seat down with the supplied bolts.

Install Arms-

- Insert arm posts into chair frame. Place flip-up arm on right side of chair. Place straight arm on left side of chair. Insert adjuster knob into threaded hole on chair frame.

Usage Instructions

Arm Adjustment-

- To adjust armrests loosen the adjuster knob $\frac{1}{4}$ turn and move the arm up or down and re tighten the adjuster knob.