



## 1202-SXL/AP

**WARNING:** MAXIMUM LOAD 800 LBS

**WARNING:** Risk of Electric Shock – Connect this furnishing to a properly grounded outlet only.

**WARNING:** Risk of fire, electric shock, or injury to persons that are associated with the use of the furnishing, such as operation or user-maintenance.

**WARNING:** For Commercial Use Only

### IMPORTANT SAFETY INSTRUCTIONS

When using an electrical furnishing, basic precautions should always be followed, including the following:

Read all instructions before using (this furnishing).

**DANGER** – To reduce the risk of electric shock:

1. Always unplug this furnishing from the electrical outlet before cleaning.

**WARNING** – To reduce the risk of burns, fire, electric shock, or injury to persons:

1. Unplug from outlet before putting on or taking off parts.
2. Close supervision is necessary when this furnishing is used by, or near children, invalids, or disabled persons.
3. Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
4. Never operate this furnishing if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the furnishing to a service center for examination and repair.
5. Keep the cord away from heated surfaces.
6. Never operate the furnishing with the air openings blocked. Keep the air openings free of lint, hair, and the like.
7. Never drop or insert any object into any opening.
8. Do not use outdoors.
9. Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
10. To disconnect, turn all controls to the off position, then remove plug from outlet.
11. For chairs and similar furnishings with movable parts such as foot supports the following statement:



**WARNING:** Risk of Injury – Keep children away from extended foot support (or other similar parts).

12. Always assist patients in and out of the chair in order to help prevent the possibility of the furnishing tipping over.

13. The arm pad is not intended to support video monitors such as televisions or computer monitors.

**WARNING:** Risk of Injury to Persons – do not place video equipment such as televisions or computer monitors on arm pads.

## GROUNDING INSTRUCTIONS

This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

**DANGER** – Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product – if it will not fit the outlet; have a proper outlet installed by a qualified electrician.



## SAVE THESE INSTRUCTIONS

### SET-UP INSTRUCTIONS

**This product is for use on a nominal 120-volt AC Single Phase, 60Hz circuit.  
Max 2.5 Amps.**

Your new chair comes completely assembled. Carefully unbox chair and slide arm post into position and tighten arm height adjustment knob.

When locating the chair make sure to provide adequate clearances around the chair; 36" in front, 12" on each side, 12" behind and 48" above the chair.

Carefully route the electrical cord from the chair to the wall outlet.

To reduce the risk of electric shock, this furnishing has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

### Operation of Chair

#### Pre-Inspection

- Notify others in the vicinity to maintain proper clearances (*see setup procedures for proper clearances*) while chair is in operation.
- Make sure the proper clearances around the chair are free and clear of any hazards.
- Inspect power cord and foot control cord to make they are in good working condition.



### **Assisting the patient into the chair**

- Make sure patient is within the stated patient weight capacity for this particular model of chair.
- Explain to the patient the steps you will be taking to ensure their safety into the chair.
- Make sure the seat is in the lowest position.
- Raise the flip-up arm allowing access to the seating area. Always assist the patient into the chair and lower the arm in front of the patient.
- Instruct the patient to place their feet on the footrest for their comfort and safety.

### **Raising the chair seat height.**

- Make sure the chair is free and clear of any patients clothing or personal belongings.
- The operator should make sure to keep safe operating distance while operating the chair.
- Turn the ACL switch located on the back of the chair to the on position.
- Depress the foot switch to raise the patient to the desired height. Additional arm adjustment can be made by lowering or raising the arm post on the side of the chair.

### **Assisting the patient out of the chair.**

- Make sure the area around and under the chair are free and clear of any hazards.
- The operator should make sure to keep safe operating distance while operating the chair.
- Depress the foot switch and lower the chair to the lowest position.
- Turn the ACL switch to the off position to lockout any accidental movement of the chair.
- Raise the flip-up arm and assist the patient out of the chair as needed.

## **Maintenance**

There are no serviceable parts. To clean the use a 10% bleach solution and with a clean white cloth. Dry completely when done.