



Power Blood Draw Chairs

Power Chair Reset

If your power chair is stuck in the up or down position and it will not respond to commands, please follow this 3-step procedure to reset your chair:

- Unplug the power cord from the wall.
- After 20 seconds, plug the power cord back into the wall.
- Hold the foot pedal with both hands and depress both the up and down controls at the same time. You will hear a series of 4-5 beeps than a long steady beep, now let go. The chair has been reset.

If your chair fails to operate after the reset, please repeat the process. *It is important that you press the up and down controls at the same time.

If your power chair is still unresponsive please contact us for further assistance.